


## Lactate measuring - what for?

- Whether for healthiness, fitness, recreation or match preparation: Only a targeted and healthy training leads to real success
- The individual optimum training level under strain can precisely be determined only by measuring the increasing lactate value in blood
- Inefficiency of training can be realized and avoided in time
- Remarkable increase in endurance performance, optimum fat burning and a more balanced activity of the heart by training control

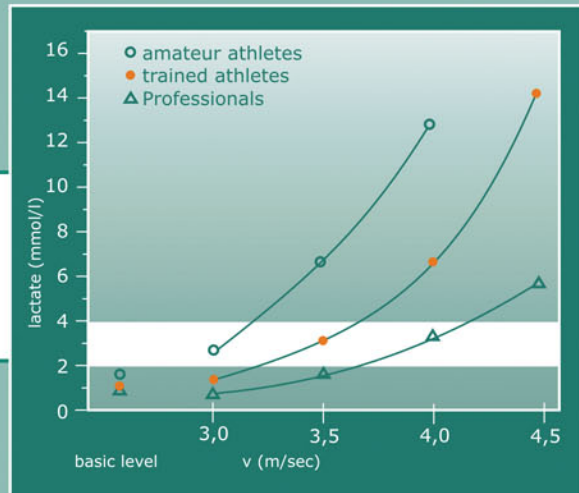

Version 5.0-10/2009  
Subject to technical modifications  
7023-9011-0771

## Products & accessories:

### Lactate Scout SOLO

Lactate Scout pocket device, 2x AAA/LR03 batteries, instruction manual

### Lactate Scout START

Lactate Scout pocket device, 2x AAA/LR03 batteries, carrying case with belt, lancing device, 25 lancets, 30 alcohol pads, 6 test strips, test solution 2,5 mL, spray bottle and instruction manual with poster, EKF Info-CD

### Big Pack 24+

24 test strips, with 25 lancets and alcohol pads

### Optional:

- Big Pack 48+ test strips
- Big Pack 72+ test strips
- lancets
- lancing device
- alcohol pads
- test solution 5 mmol/L, 10 mmol/L
- hard case
- belt case
- PC Packs EUX, USB and Bluetooth

(All offers are subject to change and errors excepted, rights for alternative offers are reserved. All customers are bound by the terms and conditions of EKF-diagnostic sales GmbH)

**Product guide on request. For further information please contact our sales team:**

#### Sales:



Sales:  
EKF - diagnostic sales GmbH  
Ebendorfer Chaussee 3  
D-39179 Barleben/Magdeburg  
Tel. +49(0)39203-785-0  
Fax +49(0)39203-785-24  
www.EKF-diagnostic.com



Producer:  
SensLab GmbH  
a Member of EKF - Group  
Bautzner Str. 67  
D - 04347 Leipzig  
Tel. +49(0)341-2341841  
Fax +49(0)341-2341840  
www.senslab.de  
www.lactatescout.com



# LACTATE SCOUT

**Performance  
under control**



## Lactate SCOUT


- The companion for effective training!
- Fast & easy to operate with test strips and comfortable scrollwheel
- Just a drop of blood (0.5 µL) is required
- Highly credible results within 10 sec.
- Measuring range: 0,5 - 25,0 mmol/L
- VC: 3 - 8% (depending on concentration)
- Memory for 250 measurements, PC-link & individual training programme

## Fast & easy measuring:

- Just pick a test strip out of the vial & insert into the Lactate Scout
- Clean, disinfect & dry the earlobe or the fingertip carefully
- Prick the area for blood withdrawal
- Hold the Lactate Scout with test strip onto the blood droplet
- Only 0,5 µL of blood will be uptaken, and the measurement occurs automatically
- After 10 sec measurement time, the result is displayed and stored
- Manifold options for step tests & Software analysis on the PC

## Can be used anywhere:

- in sports medicine diagnosis for medical supervision
- in physiotherapy & recreation
- in sports institutions, fitness studios, for sports clubs & personal coaching
- in training of individuals & teams, recreational & competitive sports
- as a flexible & inexpensive complement to laboratory analysis
- also for personal self-testing, comfortable & hygienic in application
- accredited as a medical device according to EU Directive IVD (CE 0483)

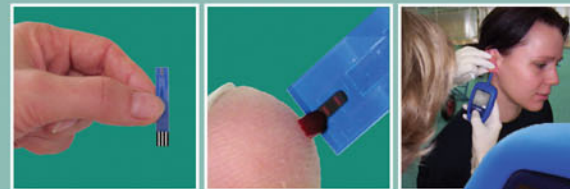

# LACTATE SCOUT

- Professional Lactate measuring fast, accurate & easy!
- Metabolism based diagnostics for optimum endurance & fitness!
- The intelligent efficiency analysis for therapy and professional sport!
- A sports-medical companion for trainers, amateurs and clubs!

